AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 WIEN
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
info@bauerfeind.at

BENELUX

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 english

Dear Valued Customer,

Thank you for choosing a product from Bauerfeind. EpiTrain is a high-quality medical product. If there is a CE mark on the label sewn into your support, this means it is a standard product. If your name is printed on the sewn-in label and there is no CE mark, this means it is a made-to-measure product specially manufactured in accordance with Section 3 No. 8 of the German Medical Devices Act. Please carefully read through the following **instructions for use**. Should you have any questions contact your medical provider, medical retailer or our customer service department.

Purpose

EpiTrain is improved active elbow support with unique function during elbow movement and physiotherapy treatment. It has specially developed, anatomically shaped viscoelastic inserts designed to compress and massage the soft tissue around the joint as the elbow moves. The enhanced circulation in the joint and promotion of lymph drainage lead to reduced swelling, pain relief and improved functionality.

 **EpiTrain must be fitted for initial use by a trained specialist, otherwise we cannot accept any liability for the product.**

Indications

- Irritated tendon – e.g., tennis elbow, golfer's elbow (epicondylopathies)
- Worn/inflamed joint (arthrosis/arthritis)
- Post-operative and post-traumatic irritation
- Muscle and tendon disorders (tendomyopathies)

Risks of using this product

 **Important Information**

- EpiTrain is a prescribed product that should be applied under a physician's guidance. EpiTrain should only be applied in accordance with these instructions for use and for the specified areas of application.
- No liability is accepted for improper use of the product.
- Changes must not be made to the product. Non compliance can reduce the performance of the product, thus no liability will be accepted.
- Should you notice any unusual changes (e.g., increased symptoms), please contact your medical provider without delay.
- A combination with other products must be discussed with your medical provider prior to use.
- The product's unique benefits are demonstrated primarily during physical activity. The product should be removed during prolonged periods of rest (e.g., sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances that include grease or acids.
- With the assumption of proper fitting, no side effects involving the body as a whole have been reported to date. Supports and orthoses¹ applied externally to the body can, if tightened excessively, lead to localized pressure or, rarely, constrict the underlying blood vessels or nerves.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

Hypersensitivity reactions harmful to health have not been reported to date. For patients experiencing the following conditions, such supports and orthoses should only be fitted and applied after consultation with your medical provider:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the arms.
3. Impaired lymph drainage – including soft tissue swelling of uncertain origin, located remotely from the fitted aid.

Application instructions

Fitting EpiTrain

1. Turn the upper part of the support inside out until the circular depression in the viscoelastic inserts are visible (fig. 1).
2. Grasp the support near the silicone inserts and pull it up the arm until the inserts are slightly below the elbow joint. The outer insert should point towards the outward bony protrusion of the epicondyle (fig. 2).
3. Unroll the upper section over the elbow ensuring that the outward bony protrusion of the epicondyle is located in the circular depression in the viscoelastic insert (fig. 3).
4. If possible, the inside bony protrusion should be located in the circular depression of the inner viscoelastic insert (fig. 4).

Removing the support

To remove EpiTrain, grasp the support on the lower end near the silicone inserts and pull down toward the hand.

Product care instructions

The sewn-in label of the EpiTrain, which contains information on the product name, size, manufacturer, washing instructions and CE mark, is located on the inside of the knitted fabric.

All Train products² can be machine washed separately at 30°C with liquid detergent. Please note the following when washing your Train products:

1. For products with Velcro® straps: Remove the straps if possible, or fasten them securely at the place provided for this purpose.
2. If possible, also remove pads/inserts and their terrycloth covers before washing.

We recommend using a mesh bag. Air dry your Train product. Do not dry it in a dryer because this can compromise the knitted fabric. Please wash your Train product regularly so that the compression of the knitted fabric can be retained. This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

Notes on reuse

The product is intended to be fitted only once and to one patient.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

The product can be disposed of without any restrictions.

Version: 2016-03

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

² Standardized washing tests show that after 50 washing machine cycles full functionality of the Train active support is retained.

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