**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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en english

These instructions for use can also be downloaded from [www.bauerfeind.com/downloads](http://www.bauerfeind.com/downloads).

## Dear Customer,

Thank you for choosing a product from Bauerfeind. By purchasing the MalleoTrain Plus, you have selected a product which meets very high medical and quality standards. Read all of these instructions for use carefully because they contain important information about the use, indications, contraindications, application, care and disposal of the MalleoTrain Plus. Keep these instructions for use in a safe place, as you may want to read them again. If you have any questions, contact your treating physician, your medical retailer or our Technical Service department.

## Purpose

The MalleoTrain Plus is an active ankle support. Its effects are felt particularly during movement, for example during sporting activities. Here, the viscoelastic pad inserts (massage pads), which are molded to the shape of the body, actively compress and massage the tissue and the ligaments. The improved flow of blood through the joint capsule and the promotion of lymph drainage lead to reduced swelling, pain relief and improved function. In addition to the support, the strap helps wearers utilize a sports taping technique. A strap helps to improve the way wearers feel the movement of

their foot. Similar to the effects of sports taping, this optimizes how the body's center of gravity is guided over the foot, helping prevent injuries and accelerating the healing process.

## Indications

- Chronic, post-traumatic or post-operative irritation of soft tissue in the ankle area
- Ligament insufficiency
- Supination prophylaxis, particularly during sports activities
- Post-operative rehabilitation

## Risks associated with use of this product

### ⚠ Caution

MalleoTrain Plus is a prescribable product. In case you have received it by prescription from your doctor or physiotherapist, the use and duration of use must be agreed with her/him before utilizing the product. In any case, instruction is required by the specialist who provided you with the product.

Consult your treating physician before using this product together with other products, e.g. compression stockings. Seek immediate medical advice if your symptoms worsen or if you notice any unusual changes. No product liability can be accepted if the product is used improperly or for any purpose than its proper, stated purpose.

The benefits of this product are felt primarily during physical activity. Take off the MalleoTrain Plus during extended rest phases (e.g. sleeping, long periods of sitting down).

Never allow the product to come into contact with substances containing fat/grease or acid, ointments or lotions.

Do not make any alterations to the product.

Non-compliance can reduce the performance of the product, so no liability can be accepted if any alterations are made.

No side effects which affect the entire organism have been reported to date. Correct usage and fitting is assumed. Do not put the MalleoTrain Plus on too tightly or it could create localized pressure symptoms. In rare cases constriction of the blood vessels and nerves is possible. If this happens, loosen the straps of the MalleoTrain Plus and possibly have the size of your MalleoTrain Plus checked.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction

## Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions, such aids should only be fitted and applied after consultation with your medical advisor:

- Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
- Impaired sensation and circulatory disorders of the legs/feet (e.g. diabetes mellitus)
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted support

## Application instructions

### Putting on and removing the MalleoTrain Plus

#### Putting on the product

- 1 Pull on the support so that the start of the strap is located on the inside of the foot. The start of the strap defines the direction in which the strap needs to be wound around the foot. When fitting the strap, please make sure there are no creases.
- 2 Bring the strap under the sole of your foot toward the outside of your foot. By pulling the strap you can stabilize the foot and ankle comfortably.
- 3 Now bring the strap diagonally across the top of the foot and back around the ankle mortise.
- 4 5 Bring the strap

diagonally forward again to the inside of the foot.

- 6 Pull the strap under the sole of your foot toward the outside of your foot and over the top of your foot.
- 7 Now secure the end of the strap by fastening its Velcro strip to the section of the strap which is underneath it. Alternatively, you can also use the designs on the support to guide you. Blue dots on the titanium-colored knit fabric indicate where the strap is supposed to go. When wrapping the strap around your foot, follow the dots knitted into the fabric, starting with one dot, going to two dots and then to three dots.

## Cleaning instructions

**Note:** Never expose the MalleoTrain Plus to direct heat (e.g. from a heater or direct sunlight, or as a result of storing it in your car etc.). This could damage the materials and possibly make the MalleoTrain Plus less effective.

Close the Velcro fastener to prevent the product from being damaged. Wash the support separately in a lingerie bag with a mild laundry detergent at 30° C. Do not run the spin cycle on your washing machine at speeds higher than 500 rpm. Wash the MalleoTrain Plus regularly so that the knit fabric retains its compressive properties.

## Information about re-use

The MalleoTrain Plus is intended for your use only, so do not give the MalleoTrain Plus to anyone else to use.

## Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

## Disposal

You can dispose of this product via the standard household waste.

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