**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 WIEN  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
info@bauerfeind.at

**BELUX**

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en english

## Dear customer,

thank you for choosing a product from Bauerfeind. ManuTrain is a high-quality medical product. Please carefully read through these **instructions for use** and if you have any questions contact your doctor, medical retailer or our Technical Service department.

## Intended use

This support is anatomically shaped to help stabilize the wrist. On the inside of the hand it has an anatomically shaped (and individually adjustable) spatula-like insert. The arrowhead-shaped pads (pressure pads) of visco-elastic silicone rubber are incorporated into the side of the support, where they apply a targeted, gentle compressive massage to the soft tissues of the joint. During movement ManuTrain massages the soft tissues to enhance blood flow and thus produce a rapid reduction in swelling and bruising. Positioning the hand in extension (after individual adjustment of the stay on the inner hand by heating), relaxes the extensor muscles of the forearm, relieving the tendon attachment at the elbow and thereby reducing irritation. The additional Velcro® fastening fully secures the palm splint thus stabilizing the wrist joint.

**⚠ ManuTrain must be fitted for the first use by a trained specialist, otherwise we cannot accept any liability for the product.**

## Indications

- Wrist wear (arthritis)
- Wrist injuries, e.g. sprains
- Other types of wrist pain (arthralgia), e.g. rheumatic
- Inflamed tendon sheath
- Instability (e.g. radioulnar subluxation)

## Risks of using this product

### ⚠ Important Information

- ManuTrain is a prescribed product that should be applied under a physician's guidance. ManuTrain should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The product should not undergo any improper modification. Failure to comply with the requirement can impair product performance and thus rule out any product liability.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical advisor without delay.
- A combination with our products must be discussed beforehand with your medical advisor.
- The product's unique action is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances including grease or acids.

**⚠** Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

## Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and worn after consultation with your medical advisor:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the hand.
3. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports or orthoses<sup>1</sup> applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

## Application instructions

### To modify the thermoplastic stay

Heat with a hot air gun at 180° C (365° F) as required.

### Fitting ManuTrain

1. Open the Velcro® fastening.
2. Now insert the separate palm stay into the opening provided **1**.
3. Slip the support down the hand so that the wrist is enclosed, pressure free, in the profile inserts **2**.
4. The support is correctly positioned when the small oval cut-out in the profile insert is over the ulnar condyle **3**.
5. Finally close the Velcro® fastening.

### Taking off the support

- Undo the Velcro strap and take off the support **4**.

## Washing instructions and product care

The sewn-in label of the ManuTrain, containing information about the product name, size, manufacturer, washing instructions and CE identification, is located on the inside of the knitted fabric.

All Train<sup>2</sup> products can be machine washed separately at 30°C with liquid detergent. Please note the following when washing your Train products:  
1. For products with Velcro® straps: Remove the straps if possible, or fasten them securely at the place provided for this purpose.  
2. If possible, also remove pads/inserts and their Terrycloth covers before washing.  
We recommend using a mesh laundry bag. (Spin cycle: max. 400 r.p.m.). Air dry your Train product. Do not dry it in a dryer because this can compromise the knitted fabric.

Please wash your Train product regularly so that the compression of the knitted fabric can be retained. This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

## Notes on reuse

The product is intended to be fitted only once and to one patient.

## Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly. Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

## Disposal

The product can be disposed of without any restrictions. If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Version: 2016-02

<sup>1</sup> Orthosis = orthopedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso  
<sup>2</sup> Standardized washing tests show that after 50 washing machine cycles full functionality of the Train® active support is retained. (Liquid detergent was used.)

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