**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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en english

**These instructions for use can also be downloaded from [www.bauerfeind.com/downloads](http://www.bauerfeind.com/downloads).**

## Dear Customer,

thank you for choosing a product from Bauerfeind. MyoTrain is a high-quality medical product. Please read through these **instructions for use** carefully. If you have any questions please contact your doctor, medical retailer or our Technical Service department.

## Intended use

MyoTrain is an active support for the treatment of muscle injuries of the thigh. MyoTrain replaces the conventional compression bandage fitted by the doctor or therapist after muscle injuries and represents an effective therapeutic aid, for example in supporting the work of the physiotherapist. Immediately after the injury, compression pads<sup>1</sup> are inserted in the support to restrict the extent of the injury and promote healing. The two compression pads can be cooled and used flexibly to match the progress of healing. At a later stage, MyoTrain can be fitted and worn without the compression pads. MyoTrain reduces the risk of a further injury, which might then require protracted treatment.

**MyoTrain must be fitted for the first use by a trained specialist, otherwise we cannot accept any liability for the product.**

## Indications

- Muscle hardening, pulled or torn muscles in the thigh
- Prevention, particularly of repeat injuries
- Support for the follow-up treatment of muscle injuries
- Muscle bruising on the thigh (thigh knock)

## Risks of using this product

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

- MyoTrain should only be worn in accordance with the instructions con-tained in these usage directions and for the listed areas of application.
- The product is indicated for the acute management of the patient and the prevention of subsequent injuries.
- No liability is accepted for improper use of the product.
- Changes must not be made to the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your doctor without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your doctor.
- The product's unique action is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Side effects, that affect the entire organism, have not been reported to date. Correct fitting is assumed. Any supports and orthoses<sup>2</sup> applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

## Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions

such aids should only be fitted and worn after consultation with your doctor:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Varicose veins.
3. Impaired sensation and circulatory disorders of the legs, e.g. diabetes mellitus.
4. Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

## Application instructions

### Initial fitting (by a trained fitter)

1. Fit MyoTrain in the standing position with your leg relaxed and slightly flexed at the knee.
2. Remove the strap system from the support.
3. Pull on the support, centering the rear velour strip (on the outside) over the in-jured muscle area. The whole of the injured area must be covered by the support (Fig. 1).
4. Roll back the top edge of the support approx. 2/3 distally (down) (Fig. 2).
5. Place the large pad (functional pad) over the injured site on the thigh. The smaller pad (acute pad) may optionally be inserted for increased pressure (Fig. 3).
6. Unroll the support edge back to its correct position.
7. Accurately fasten the Velcro tapes to the rear velour strip. The straps must pass from the bottom at the back to the top at the front – short straps at the bottom, long straps at the top – (Fig. 4!)
8. Attach the counter-traction system (eyelets) to the front velour strip (Fig. 5), ensuring that the straps pass smoothly through the eyelets. Starting with the lower pair of straps, tighten the inner and outer straps at the same time and fasten (Fig. 6). Adjust the support and strap pressure with the knee flexed slightly so that the patient can feel the compression pressure but no pain.

Once the basic fitting has been performed correctly as described above, the support can subsequently be fitted and removed easily simply by opening and closing the straps. Be careful to ensure that the basic positioning for the course of the straps is not modified.

### Use of the pad

The following sketches show you the various options for inserting the cushions depending on the stage of treatment:

1. Immediately after the injury:  
Locate the acute pad directly over the injured site. Position the functional pad on top of the acute pad. Both pads can be pre-cooled. The acute pad can be removed after approx. 24 hours (Fig. 3). After this time the increased compression produced by the acute pad is no longer required.
2. Subsequently:  
The functional pad can continue to be worn subsequently, particularly on the resumption of training. If necessary, you can remove the side panels of the functional pad or fold them in on both sides (Fig. 7).
3. MyoTrain can be worn without pads.

## Product care

The sewn-in label of the MyoTrain, which contains information on the product name, size, manufacturer, washing instructions and CE mark, is located on the upper edge of the support.  
Wash MyoTrain separately by hand at 30°C (86°F) with a mild detergent. To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

## Notes on reuse

MyoTrain is intended for your treatment only and has been specifically selected according to your measurements. Do not pass your MyoTrain on to other people.

## Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly. Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

## Disposal

You can dispose of the MyoTrain in accordance with national legal requirements.

If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Version: 2017-04

<sup>1</sup> Pad = pressure pad

<sup>2</sup> Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

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