AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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(en) english

Dear patient,

thank you for choosing a product from Bauerfeind® Orthopedics. SofTec Lumbo is a high-quality medical product. Please read carefully through these **instructions for use**. If you have any questions please contact your doctor, your medical retailer or our Technical Service department.

Intended purpose

SofTec Lumbo is a multifunctional lumbar orthosis¹ for stabilization of the lumbar spine and the lumbosacral transition. Its modular design permits controlled treatment according to an individually adaptable multi-stage regimen.

Stage 1 (Stabilization phase):

For effective „postoperative“, or initially conservative, stabilization of the spine, the support is used together with the shell.

Stage 2 (Mobilization phase):

During progressive rehabilitation, during the initial stages of mobilization, the shell is removed and the knitted section supporting the affected lumbosacral area is reinforced with CFR (carbon-fiber reinforced) plastic rods.

Stage 3 (Activity phase):

As patients become increasingly active, the CFR rods are replaced by corset rods, thereby reducing external support.*

⚠ SofTec Lumbo must be individually adapted to the patient by a trained orthopaedic technician in order to gain the maximum effect and optimal wear comfort from this orthosis. **If the product is not used properly any claim for warranty will expire.**

Indications

- After lumbar disc surgery
- After a stabilizing/stiffening operation (e.g. spondylodesis)
- Degenerative instability

- Vertebral displacement (spondylolisthesis) due to a vertebral arch defect (spondylosis)
- Irritation of the small intervertebral joints (facet syndrome)
- Constriction of the lumbar spinal canal (lumbar spinal stenosis)
- Symptomatic constriction of the nerve outlets in the lumbar spine (symptomatic stenoses in the intervertebral foramina)
- Bone-destroying conditions, e.g. cancer (tumor, metastasis) or inflammatory conditions
- Vertebral fractures of the lower lumbar spine

Risks of using this product

⚠ Important information

- SofTec Lumbo is a prescribed product that should be worn under a physician´s guidance. SofTec Lumbo should only be worn in accordance with the instructions contained in these usage directions and for the listed areas of application.
- SofTec Lumbo is intended for single use by the patient.
- The plastic shell and rods of SofTec Lumbo may only be modified by trained specialists. Failure to comply with this requirement may adversely affect product performance, thereby cancelling any product liability.
- If the knitted sections of SofTec Lumbo are subsequently modified, the proper functioning of the support can no longer be guaranteed.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your doctor without delay.
- Please consult your doctor before using SofTec Lumbo in combination with other products.
- Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.
- SofTec Lumbo can readily be disposed of with domestic refuse.
- Side effects, with the assumption of correct fit involving the body as a whole have not been reported to date. Any supports or orthoses applied externally to the body can, if tightened excessively, signs of localized pressure, rarely, constrict the underlying blood vessels or nerves.

⚠LATEX Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Take off the orthose if you notice any allergic reaction.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such supports or orthoses should only be fitted and worn after consultation with your doctor:

- Skin disorders / injuries to relevant parts of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Impaired sensation.
- Severe restriction in the performance of heart and lungs (risk of elevated blood pressure if the aid is worn during severe physical exertion).

Application instructions

Fitting and removing SofTec Lumbo

Despite its special design, SofTec Lumbo is always fitted and removed in the same way at all stages.

Fitting:

- After it has been individually adjusted by the orthopaedic technician, the shell must sit loosely on the iliac crests (Fig. 2). When fitting the orthosis without the shell, ensure that the support is correctly positioned. If necessary, practice correct fitting with the orthopaedic technician.
- Place the support around the trunk so that the reinforcements visible on the outer knitted section are positioned parallel to the spine (Fig. 3).
- Place your fingers in the specially provided loops on the two halves of the light grey fastener and pull evenly forward (Fig. 4).
- Press the left half of the fastener against your stomach and place the right half above it to secure the Velcro fastening. If the optional pressure pad for the abdominal wall is used, ensure that this is posi-tioned correctly (this pressure pad should be centrally positioned beneath the fastening).
- When securing the Velcro fastening, remove your fingers from the left loop first, followed by the right loop (Fig. 5).
- Take hold of the two LPT (**Low Power Tension**) straps in both hands and pull evenly forward (Fig. 6). Secure both LPT straps onto the large fastening (Fig. 7). With the „long“ version, always fasten the lower straps first and then secure the upper straps on top.
- When the support is first fitted, the optimal, individual fit must be checked, and if necessary corrected, by a trained fitter. Practice fitting the support correctly.

Removal:

- Undo the Velcro fastenings of the LPT straps on the outer knitted section and pull back the straps to loosen the support and fasten the ends of the straps to the retracted sections.
- Open the large light grey fastener.
- Remove the support and reclose the light grey fastener.

Product care

⚠ Never expose the orthosis to direct heat (e.g. heater, sunlight, etc.)!

- SofTec Lumbo can be washed separately by hand at 30° C (86°F) using a mild detergent. Ensure that the plastic shell is removed beforehand. Proceed as follows:
- Undo the Velcro fastenings of the two thin elastic straps. Pull the straps out of the holes and remove.
- Undo the two Velcro fastenings between the outer and inner knitted sections located at the center of the back.
- Remove the plastic shell from the support.
- The shell can easily be washed with a sponge, using a pH-neutral soap or shampoo. Do not dry using direct heat (dryer, heater). Either rub dry with a cloth or air dry.
- Removal of the CFR or corset rods is not necessary.
- If the rods are to be removed from the outer knitted section, turn back the fasteners at the top (red section) and push out the rods (Fig. 8).
- To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro fastenings before washing (including the small Velcro dots between the inner and outer knitted sections).
- After washing, perform the above sequence in reverse. When re-inserting the CFR rods, check the markings for correct orientation (top / bottom and left / right). Also check the correct positioning of the knitted sections containing the shell. Fasten identically coloured Velcro dots to each other (the sewn-in labels are always at the top, the shell is marked towards the top).

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

Notes on reuse

The SofTec Lumbo is intended for your sole use and has been individually adapted for you. For this reason, do not pass the SofTec Lumbo on to other people.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

You can dispose of the SofTec Lumbo in accordance with national legal requirements.

Version: 2016-12

^[1] Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

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