AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 WIEN
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
info@bauerfeind.at

BENELUX

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2

Dear customer,

thank you for placing your trust in Bauerfeind. By purchasing the VenoTrain glider plus donning aid, you have selected a product that meets very high medical and quality standards. The excellent workmanship provides you with the convenience you require to put on your compression stockings quickly and precisely, day after day.

Please read through the instructions for use carefully, because it contains important information on using your donning aid. Please keep the instructions for use safe, as you may want to read it again later. If you have any questions, please contact your physician or medical retailer.

Intended use

VenoTrain glider plus is a donning aid. It makes it easier to put on compression stockings (up to compression class 3) for the lower extremities.

Indications

- Restricted mobility
- Paralysis symptoms
- Results of injury
- Joint disorders with inflammation or wear
- Chronic back pain
- Osteoporosis (reduction in bone density)
- Muscular imbalances in fingers and hands
- Deformations of the hands caused by illness
- Morbid obesity
- Stiffness in the knee, hip, and spine area

Risks of using this product

 **Caution¹**
The VenoTrain glider plus donning aid must only be used in accordance with the instructions in this user guide and the intended use given. There are no known risks if the product is used properly. Compression stockings with a compression class of more than 3 can cause the donning aid to become deformed. Incorrect storage or direct heat can heat up and change the surface of the donning aid, which can lead to injury.

Instructions for use

1 Place the VenoTrain donning aid on a firm surface with the opening of the semicircular basket pointing forward.
Straighten out the compression stocking and hold it in the basket with the toe pointing forward and the heel pointing toward you.
Fold the top band of the compression stocking over the basket with both hands and pull the material over the top half of the basket until the heel appears at the upper edge of the frame. Please make sure that the stocking is pulled straight and is not twisted.

Tip: Insert your hand into the toe to ensure that it is in the perfect position.

2 Now unfold the handles until they are upright.

3 Position the donning aid on the floor in front of you. To put on your stockings safely, we recommend that you use the donning aid sitting down. Now slide your toes into the foot of the stocking that you have opened up.
Push your foot through until you can place it firmly on the floor. Please check that the toe and heel of the compression stocking are in the correct position.

4 Clasp the handles and slowly pull the donning aid toward your knee.
When you have reached the bottom of your knee, tilt the donning aid backward, remove it from your leg, and put it to one side.

5 Finally, gradually pull the compression stocking upward with both hands and spread out the material evenly. Please make sure that you put on the stocking carefully and do not pull it hard.² Gently smooth down the stocking on the leg and avoid creating creases.
Tip: Use our VenoTrain rubber gloves for a light and gentle grip.

Cleaning instructions

Please use a damp cloth or standard disinfectant to clean the donning aid.

Maintenance information

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The VenoTrain glider plus is supplied ready for use and can be used immediately.

Technical specifications/parameters, accessories

The donning aid is a metal wire frame.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2016-02

Our quality promise

This item was checked thoroughly before being packed. However, should you have any complaints, please return the product to your medical retailer.

¹ Note on the risk of personal injury (risk of injury, harm to health, accidents) or damage to property.

² Please also take note of the instructions for use for your compression stockings.

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Für das vorliegende Produkt erklärt die Bauerfeind AG die Konformität zur Richtlinie 93/42/EWG über Medizinprodukte.